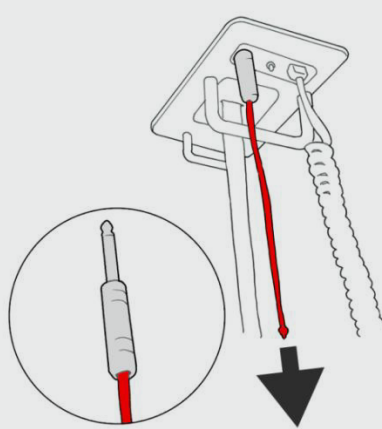
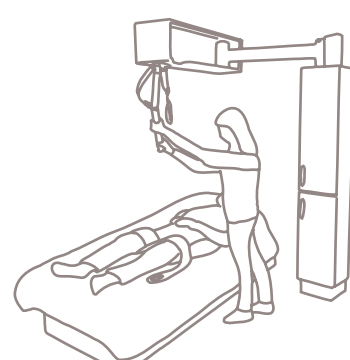
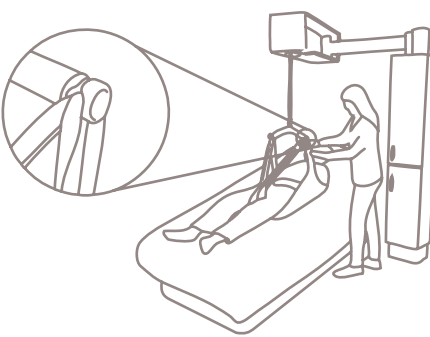
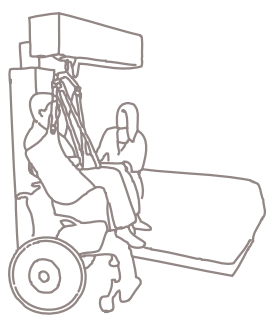
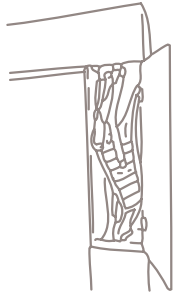
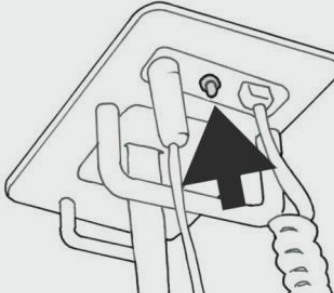
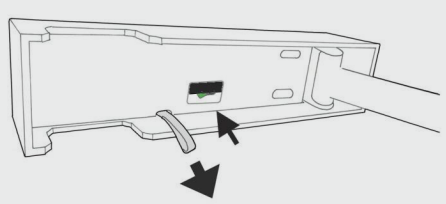


# Integralift

## Quick User Guide

|                                                                                                                                                                                                                                                                                                                                                                                                                                                                                         |                                                                                                                                                                                                                                                                                |                                                                                                                                                                                                                                                                 |
|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <p><b>HOW TO USE THE INTEGRALIFT HOIST:</b></p>                                                                                                                                                                                                                                                                                                                                                                                                                                         |                                                                                                                                                                                                                                                                                | <p><b>EMERGENCY STOP:</b></p>  <p>Pull the red string.</p>                                                                                                                   |
| <p><b>1</b></p> <p><b>ACTIVATE</b></p> <p>Open closet and pull out the lift.<br/>Position the lift above the user.</p>                                                                                                                                                                                                                                                                                 | <p><b>2</b></p> <p><b>PLACE THE SLING</b></p> <p>Position the sling under the user.<br/>Hook the sling on the lifting bar.</p>                                                               |                                                                                                                                                                                                                                                                 |
| <p><b>3</b></p> <p><b>THE TRANSFER</b></p> <p>Lift and position the user over the chair.<br/>Lower the user into the chair.</p>                                                                                                                                                                                                                                                                       | <p><b>4</b></p> <p><b>COMPLETED</b></p> <p>Place the lifting bar and the remote in their designated places. Swing the lift back to its wall position. Put the sling back in the closet.</p>  | <p><b>EMERGENCY LOWERING:</b></p>                                                                                                                                          |
| <p><b>SAFE WORKING LOAD:</b></p> <p><b>230kg</b></p>                                                                                                                                                                                                                                                                                                                                                                                                                                    |                                                                                                                                                                                                                                                                                |                                                                                                                                                                                                                                                                 |
| <p><b>MAINTENANCE</b></p> <ul style="list-style-type: none"> <li>Read User Manual</li> </ul>                                                                                                                                                                                                                                                                                                         | <p><b>SIMPLE TROUBLESHOOTING:</b></p> <ul style="list-style-type: none"> <li>Battery charger is connected to an electric socket?</li> <li>Remote control connected?</li> <li>Emergency stop connected?</li> </ul>                                                              | <p>If the lifter should stop working, press the emergency lowering button until the user is lowered safely on the bed or chair</p>                                                                                                                              |
| <p><b>MECHANICAL EMERGENCY LOWERING:</b></p>  <p>If the electrical emergency lowering fails, use mechanical lowering: On the rear of the top cabinet there is a small window with a switch (stand on a footstool).</p> <ol style="list-style-type: none"> <li>1. Flick the switch from Green to Black.</li> <li>2. Pull the black handle intermittent, keeping the user at a secure speed.</li> </ol> |                                                                                                                                                                                                                                                                                | <p><b>SAFETY:</b></p>  <ul style="list-style-type: none"> <li>Read the User Manual</li> <li>Make sure the sling loops are properly attached to the spreader bar</li> </ul> |